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GARDNERELLA VAG. AGAR (Sheep Blood 5%)

Selective medium for *Gardnerella vaginalis* isolation.

TYPICAL FORMULA	(g/l)
Peptospecial	23.0
Starch	1.0
Sodium Chloride	5.0
Gentamicin	6.0 mg
Amphotericin B	2.0 mg
Nalidixic Acid	30.0 mg
Agar	14.0
Sheep Defibrinated blood	50.0 ml
Final pH 7.3 ± 0.2	

DESCRIPTION

GARDNERELLA VAG. AGAR (Sheep Blood 5%) is a selective medium for *Gardnerella vaginalis* isolation.

PRINCIPLE

GARDNERELLA VAG. AGAR (Sheep Blood 5%) represents an optimal culture medium for fastidious microorganisms. Peptospecial is a source of nitrogen, carbon, amino acids and vitamins. Gentamicin inhibits many Gram-positive and Gram-negative bacteria (enterobacteria, staphylococci, *Pseudomonas*), Nalidixic Acid inhibits Gram-negative bacteria whilst Amphotericin B inhibits the development of fungi and yeasts. Blood is a further growth supplement and allows to enhance haemolytic reactions.

TECHNIQUE

Inoculate the plates streaking the specimen onto the medium surface, using a sterile loop. Incubate at 36±1°C for 18-48 hours in a 5-10% CO₂ atmosphere.

INTERPRETATION OF RESULTS

Examine plates for growth and hemolytic reactions. Four types of hemolysis on blood agar media can be described:

1. α-hemolysis is the reduction of hemoglobin to methemoglobin in the medium surrounding the colony, causing a greenish decolorization of the medium.
2. β-hemolysis is the lysis of red blood cells, producing a clear zone surrounding the colony.
3. γ-hemolysis indicates no destruction of red blood cells and no change in the color of the medium.
4. δ-hemolysis indicates a partial lysis.

STORAGE AND TRANSPORT CONDITIONS

2-8°C away from light, until the expiry date on the label. However, our stability studies have shown that the transport at 18-25°C for 4 days, or at 35-39°C for 48 hours, does not alter in any way the performance of the product. Eliminate if signs of deterioration or contamination are evident.

WARNING AND PRECAUTIONS

The product does not contain hazardous substances in concentrations exceeding the limits set by current legislation and therefore is not classified as dangerous. It is nevertheless recommended to consult the safety data sheet for its correct use. The product is designed for *In vitro* diagnostic use and must be used only by properly trained operators.

DISPOSAL OF WASTE

Disposal of waste must be carried out according to the national and local regulations in force.

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GARDNERELLA VAG. AGAR (Sheep Blood 5%)

PRESENTATION

Ready to use plates (90 mm) containing 22+/-1 ml of medium

STORAGE

2-8°C

PACKAGING

Ref.	Content	Packaging
11054	20 plates	10 plates in thermally soldered film 2 x 10 plates in cardboard box
11054*	100 plates	10 plates in thermally soldered film 10 piles (10 x 10 plates) in cardboard box

pH OF THE MEDIUM

7.3 ± 0.2

USE

GARDNERELLA VAG. AGAR (Sheep Blood 5%) is a selective medium for *Gardnerella vaginalis* isolation

TECHNIQUE

Refer to technical sheet of the product

APPEARANCE OF THE MEDIUM

Cherry red medium, opaque

SHELF LIFE

75 days

QUALITY CONTROL

- Control of general characteristics, label and print
- Sterility control
7 days at 22 ± 1°C, in aerobiosis
7 days at 36 ± 1°C, in aerobiosis
- Microbiological control
Inoculum for productivity: 10-100 CFU/ml
Inoculum for selectivity: 10⁴-10⁵ CFU/ml
Inoculum for specificity: ≤10⁴ CFU/ml
Incubation Conditions: 18-24 hours at 36 ± 1°C, in 10% CO₂ atmosphere

Microorganism		Growth	Features
<i>Escherichia coli</i>	ATCC 25922	Inhibited	---
<i>Gardnerella vaginalis</i>	ATCC 14018	Good	β- hemolysis

TABLE OF SYMBOLS

LOT	Batch code	IVD	In vitro Diagnostic Medical Device		Manufacturer		Use by		Fragile, handle with care
REF	Catalogue number		Temperature limitation		Contains sufficient for <n> tests		Caution, consult instructions for use		Do not reuse

LIOFILCHEM® S.r.l.

Via Scozia, Zona Ind.le - 64026, Roseto degli Abruzzi (TE) - ITALY
Tel +39 0858930745 Fax +39 0858930330 Website: www.liofilchem.net E-mail: liofilchem@liofilchem.net