CLOSTRIDIUM AGAR (Sheep blood 5%)

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CLOSTRIDIUM AGAR (Sheep blood 5%)

Selective medium for the isolation of *Clostridium difficile*.

TYPICAL FORMULA	(g/l)
Peptospecial	40.0
Disodium Hydrigen Phosphate	5.0
Sodium Chloride	2.0
Potassium Dihydrogen Phosphate	1.0
Magnesium Sulphate	0.1
Fructose	6.0
Agar	15.0
D-Cycloserine	0.25
Cefoxitin	8.0 mg
Defibrinated Sheep Blood	50 ml
Final pH 7.4 ± 0.2	

DESCRIPTION

CLOSTRIDIUM AGAR (Sheep Blood 5%) is a selective medium for *Clostridium difficile* isolation.

PRINCIPLE

Peptospecial is a source of amino acids and proteins. Sodium chloride maintains the osmotic balance of the medium. Sodium and potassium phosphates provide buffering capability. Fructose is a energy source. Agar is the solidifying agent. The addition of blood provides further growth factors for fastidious microorganisms. The addition of D-cycloserine and cefoxitin allows the inhibition of a large variety of microorganisms included enterobacteria, faecal enterococci, anaerobic non-spore forming gram-negative bacilli and clostridia, except *C. difficile*.

TECHNIQUE

Direct faces inoculation

Lightly inoculate the medium with the faecal sample by spreading part of the initial suspension in order to obtain isolated colonies and incubate at 36±1°C for 24-48 hours in anaerobic atmosphere.

Alcohol shock treatment

Mix equal parts of absolute or 95% ethyl alcohol and the faecal suspension. Homogenise using a vortex mixer. Leave at room temperature for 1 hour. Inoculate onto CLOSTRIDIUM AGAR (Sheep Blood 5%) and incubate at 36±1°C 24-48 hours anaerobically.

INTERPRETATION OF RESULTS

C. difficile grows after 48 hours of incubation at 37°C in anaerobic conditions, with 4-6 mm diameter, irregular, raised, opaque, grey-white colonies.

STORAGE AND TRANSPORT CONDITIONS

2-8°C away from light, until the expiry date on the label. However, our stability studies have shown that the transport at 18-25°C for 4 days, or at 35-39°C for 48 hours, does not alter in any way the performance of the product. Eliminate if signs of deterioration or contamination are evident.

WARNING AND PRECAUTIONS

The product does not contain hazardous substances in concentrations exceeding the limits set by current legislation and therefore is not classified as dangerous. It is nevertheless recommended to consult the safety data sheet for its correct use. The product is designed for *In vitro* diagnostic use and must be used only by properly trained operators.

DISPOSAL OF WASTE

Disposal of waste must be carried out according to the national and local regulations in force.

REFERENCES

1. Levett (1985) J. Clin. Pathol. 38: 233-234.
2. Barlett, J.G. et al. (1978) N. Eng. J. Med., **298**, 531.
3. Boriello, S.P. et al (1981) J. Antimicrob. Chemother. 7 Supp. A. 53-62.
4. George, R.H. et al (1976) J. Clin. Microbiol. **6**, 214-219.

LIOFILCHEM® S.r.l.

Via Scozia, Zona Ind.le - 64026, Roseto degli Abruzzi (TE) - ITALY
Tel +39 0858930745 Fax +39 0858930330 Website: www.liofilchem.net E-mail: liofilchem@liofilchem.net

PRODUCT SPECIFICATIONS

NAME

CLOSTRIDIUM AGAR (Sheep Blood 5%)

PRESENTATION

Ready to use plates (90 mm) containing 22+/-1 ml of medium

STORAGE

2-8°C

PACKAGING

Ref.	Content	Packaging
11060	20 plates	10 plates in thermally soldered film 2 x 10 plates in cardboard box
11060*	100 plates	10 plates in thermally soldered film 10 piles (10 x 10 plates) in cardboard box

pH OF THE MEDIUM

7.4 ± 0.2

USE

CLOSTRIDIUM AGAR (Sheep Blood 5%) is a selective medium for *Clostridium difficile* isolation

TECHNIQUE

Refer to technical sheet of the product

APPEARANCE OF THE MEDIUM

Cherry red medium

SHELF LIFE

60 days

QUALITY CONTROL

- Control of general characteristics, label and print
- Sterility control
7 days at 22 ± 1°C, in aerobiosis
7 days at 36 ± 1°C, in aerobiosis
- Microbiological control
Inoculum for productivity: 10-100 CFU/ml
Inoculum for selectivity: 10⁴-10⁵ CFU/ml
Inoculum for specificity: ≤10⁴ CFU/ml
Incubation Conditions: 24 hours up to 7 days at 36 ± 1°C

Microorganism

Escherichia coli ATCC® 25922 Inhibited
Clostridium difficile ATCC® 11204 Good

Growth

TABLE OF SYMBOLS

LOT	Batch code	IVD	<i>In vitro</i> Diagnostic Medical Device		Manufacturer		Use by		Fragile, handle with care
REF	Catalogue number		Temperature limitation		Contains sufficient for <n> tests		Caution, consult instructions for use		Do not reuse

LIOFILCHEM® S.r.l.

Via Scozia, Zona Ind.le - 64026, Roseto degli Abruzzi (TE) - ITALY
Tel +39 0858930745 Fax +39 0858930330 Website: www.liofilchem.net E-mail: liofilchem@liofilchem.net