BURKHOLDERIA CEPACIA SELECTIVE AGAR (BCSA)

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BURKHOLDERIA CEPACIA SELECTIVE AGAR (BCSA)

Selective medium for the isolation of *Burkholderia cepacia* from clinical and non-clinical specimens.

TYPICAL FORMULA	(g/l)
Meat Extract	5.0
Yeast Extract	4.0
Ammonium Sulfate	1.0
Magnesium Sulfate	0.2
Ferrous Ammonium Sulfate	0.01
Bile Salts	1.5
Potassium Dihydrogen Phosphate	4.4
Sodium Phosphate	1.4
Sodium Pyruvate	7.0
Phenol Red	0.02
Crystal Violet	0.001
Agar	15.0
Ticarcillin	0.1
Polymyxin B	150000 IU
Gentamicin	0.005
Final pH 6.2 ± 0.2	

DESCRIPTION

BURKHOLDERIA CEPACIA SELECTIVE AGAR (BCSA) is a selective medium for the isolation of *Burkholderia cepacia* from clinical and non-clinical specimens.

PRINCIPLE

The medium contains a variety of proteic substrates derived from enzymatic digestion, inorganic salts and other nutrients which satisfy nutritional requirements of these microorganisms. Crystal violet and bile salts inhibit Gram-positive cocci while Ticarcillin, Polymyxin B and Gentamicin inhibit also Gram-negative microorganisms. Alkaline end products from the metabolism of pyruvate raise the pH of the medium causing the color of the indicator (phenol red) to change from light orange to pink-red in the growth area.

TECHNIQUE

Take the sample and, if necessary, dilute 1:2 the sample in Ringer's Solution (Ref. 81059). Streak onto a BURKHOLDERIA CEPACIA SELECTIVE AGAR (BCSA) plate and incubate at 36 ± 1°C for 48-72 hours.

INTERPRETATION OF RESULTS

Typical colonies of *Burkholderia cepacia* are circular, vary from grey to sage green, with the medium changing from orange to bright pink.

STORAGE AND TRANSPORT CONDITIONS

6-12°C away from light, until the expiry date on the label. However, our stability studies have shown that the transport at 18-25°C for 4 days, or at 35-39°C for 48 hours, does not alter in any way the performance of the product. Eliminate if signs of deterioration or contamination are evident.

WARNING AND PRECAUTIONS

The product does not contain hazardous substances in concentrations exceeding the limits set by current legislation and therefore is not classified as dangerous. It is nevertheless recommended to consult the safety data sheet for its correct use. The product is designed for *In vitro* diagnostic use and must be used only by properly trained operators.

DISPOSAL OF WASTE

Disposal of waste must be carried out according to the national and local regulations in force.

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BURKHOLDERIA CEPACIA SELECTIVE AGAR (BCSA)

PRESENTATION

Ready plates (90 mm) containing 20 ± 1 ml of medium

STORAGE

6-12°C

PACKAGING

Code	Content	Packaging
11506	20 plates	10 plates in thermally soldered film 2 x 10 plates in cardboard box

pH OF THE MEDIUM

6.2 ± 0.2

USE

BURKHOLDERIA CEPACIA SELECTIVE AGAR (BCSA) is a selective medium for the isolation of Burkholderia cepacia from clinical and non-clinical specimens

TECHNIQUE

Refer to technical sheet of the product

APPEARANCE OF THE MEDIUM

Orange coloured medium, slightly opalescent

SHELF LIFE

6 months

QUALITY CONTROL

- Control of general characteristics, label and print
- Sterility control
7 days at $22 \pm 1^\circ\text{C}$, in aerobiosis
7 days at $36 \pm 1^\circ\text{C}$, in aerobiosis
- Microbiological control
Inoculum for productivity: 10-100 UFC/ml
Inoculum for selectivity: 10^4 - 10^5 UFC/ml
Inoculum for specificity: $\leq 10^4$ UFC/ml
Incubation Conditions: 48-72 h at $36 \pm 1^\circ\text{C}$, in aerobiosis

Microorganism		Growth	Features
<i>Burkholderia cepacia</i>	ATCC 25608	Good	Grey colonies with bright pink medium*
<i>Burkholderia cepacia</i>	ATCC 24416	Good	Sage green colonies with bright pink medium*
<i>Pseudomonas aeruginosa</i>	ATCC 27853	Inhibited	---

* low numbers of colonies may not produce a colour change of the medium

TABLE OF SYMBOLS

LOT	Batch code	IVD	In vitro Diagnostic Medical Device		Manufacturer		Use by		Fragile, handle with care
REF	Catalogue number		Temperature limitation		Contains sufficient for <n> tests		Caution, consult instructions for use		Do not reuse

LIOFILCHEM® S.r.l.

Via Scozia, Zona Ind.le - 64026, Roseto degli Abruzzi (TE) - ITALY
Tel +39 0858930745 Fax +39 0858930330 Website: www.liofilchem.net E-mail: liofilchem@liofilchem.net