Helicobacter pylori Test

Selective medium for screening and detection of
Helicobacter pylori from clinical specimens.

DESCRIPTION

Helicobacter pylori Test is a selective enrichment broth used for detection of *Helicobacter pylori* from biptic and faecal samples.

TYPICAL FORMULA* (Per litre of Purified Water)

Peptocomplex	26.0 g
Dextrose	1.0 g
Sodium Chloride	5.0 g
Vitalex	10.0 ml
Urea	30.0 g
Phenol Red	0.03 g
Antimicrobial Mix	0.025
Final pH 6.0 ± 0.2 at 25°C	

*Adjusted and/or supplemented as required to meet performance specifications.

METHOD PRINCIPLE

Helicobacter pylori is a gastroduodenal pathogen, the etiological agent of diseases such as peptic ulcer and gastritis, also described as a risk factor in the onset of gastric cancer. The organism is found in the mucous layer of the stomach overlying the gastric epithelium. Accurate detection of *H. pylori* can also be carried out in faeces, in particular for very young paediatric patients, who should preferentially be investigated by noninvasive methods.

The antimicrobial agents included in the medium inhibit the contaminating flora allowing the selective growth of target organism. Detection of *H. pylori* is based on the urease activity, showed by the colour change of the broth from yellow to red-fuchsia.

TEST PROCEDURE

Biopsy specimens: The biopsies for Helicobacter Pylori Test can be taken during gastroscopy. Biopsy an area of normal-looking tissue rather than an area affected by erosions or ulceration. This is because *H. pylori* may be present in smaller numbers if the epithelium is eroded or the mucous layer is denuded. The standard biopsy forceps will provide a specimen of sufficient size (2 - 3 mm diameter). Be careful not to contaminate the specimen with blood as it might interfere with the test. Place the biopsy specimen into a tube of Helicobacter Pylori Test so that it is totally immersed into the broth.

Stool specimens: Homogenize about 1-2 g faeces in 5 ml physiological solution. Add 4-5 drops specimen suspension to a test tube.

Once inoculated, incubate the tubes of Helicobacter Pylori Test at 37°C for 18-24 hours.

Notes: Patient should stop taking proton pump inhibitor (PPI) 2 weeks and antibiotic 4 weeks before exam to avoid false negative results. Sometimes, the presence of other urease producing pathogens in stomach may cause false positive results.

INTERPRETING RESULTS

Colour change of the broth from yellow to red-fuchsia is to be considered a positive result (urease production). Subculture onto a selective or non selective agar medium. Final identification should be done performing appropriate biochemical tests.

APPEARANCE

Clear, yellow.

STORAGE

Store at 2-8°C away from light. Do not use the product beyond its expiry date on the label or if product shows any evidence of contamination or any sign of deterioration.

SHELF LIFE

1 year.

QUALITY CONTROL

The medium is inoculated with the microbial strains indicated in the QC table.

Incubation: 18-24 h / $37 \pm 1^{\circ}\text{C}$

QC Table.

Strain		Growth	Color of the medium
<i>Helicobacter pylori</i>	ATCC® 43504	Good	Red-fuchsia
<i>Escherichia coli</i>	ATCC® 25922	Inhibited	Yellow

WARNING AND PRECAUTIONS

The product does not contain hazardous substances in concentrations exceeding the limits set by current legislation and therefore is not classified as dangerous. It is nevertheless recommended to consult the safety data sheet for its correct use. The product is intended for *in vitro* diagnostic use only and must be used by properly trained operators.

DISPOSAL OF WASTE

Disposal of waste must be carried out according to national and local regulations in force.

BIBLIOGRAPHY

See the references at the end of this document.

TABLE OF SYMBOLS

See the table of symbols at the end of this document.

Product Description	Format	Packaging	Ref.
Helicobacter pylori Test	Tube	20 tubes x 2.5 ml	20090

This document is available from the online Support Center:

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