Streptococcus Broth

Selective liquid medium for detection of streptococci from clinical specimens containing a mixed flora.

TYPICAL FORMULA	(g/l)
Pancreatic Digest of Casein	15.0
Papaic Digest of Soybean Meal	5.0
Sodium Chloride	4.0
L-Cystine	0.2
Sodium Sulphite	0.2
Glucose	5.0
Sodium Azide	0.2
Sodium Citrate	1.0
Crystal Violet	0.002
Final pH 7.4 ± 0.2	

DESCRIPTION

Streptococcus Broth is a liquid medium used for the selective isolation and cultivation of streptococci including group A β -haemolytic strains.

PRINCIPLE

Pancreatic digest of casein and papaic digest of soybean meal provide carbon, nitrogen, vitamins and minerals for organisms growth. Sodium chloride maintains the osmotic balance of the medium. L-cystine and sodium sulphite are incorporated to stimulate growth. Glucose is the fermentable carbohydrate. Sodium azide, sodium citrate and crystal violet are selective agents suppressing the contaminant microbial flora.

TECHNIQUE

Inoculate tubes directly with the clinical sample. Incubate at $35 \pm 2^\circ\text{C}$ for 18-24 h.

INTERPRETATION OF RESULTS

Turbidity indicates microbial growth.

Subculture to blood Columbia agar (ref. 11025) to assess the haemolytic reactions and to Todd Hewitt Broth (ref. 24111) for serological identification.

STORAGE

10-25°C away from light, until the expiry date on the label or until signs of deterioration or contamination are evident.

WARNING AND PRECAUTIONS

The product does not contain hazardous substances in concentrations exceeding the limits set by current legislation and therefore is not classified as dangerous. It is nevertheless recommended to consult the safety data sheet for its correct use. The product is designed for *in vitro* diagnostic use only and must be used by properly trained operators.

DISPOSAL OF WASTE

Disposal of waste must be carried out according to the national and local regulations in force.

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Streptococcus Broth

PRESENTATION

Glass tubes containing 10 ml of medium

STORAGE

10-25°C

PACKAGING

Ref.	Content	Packaging
24412	20 x 10 ml tubes	20 tubes in cardboard box

pH OF THE MEDIUM

7.4 ± 0.2

USE

Streptococcus Broth is a liquid medium used for the selective isolation and cultivation of streptococci including group A β-haemolytic strains

TECHNIQUE

Refer to technical sheet of the product

APPEARANCE OF THE MEDIUM

Appearance: clear to slightly opalescent

Colour: amber

SHELF LIFE

2 years

QUALITY CONTROL

- Control of general characteristics, label and print
- Sterility control
7 days at 22 ± 2°C, in aerobiosis
7 days at 35 ± 2°C, in aerobiosis
- Microbiological control
Inoculum for productivity: ≤100 CFU
Inoculum for selectivity: >10³ CFU
Incubation Conditions: 35 ± 2°C for 18-24 h

Microorganism

<i>Streptococcus pyogenes</i>	ATCC® 19615	Good
<i>Enterococcus faecalis</i>	ATCC® 29212	Good
<i>Escherichia coli</i>	ATCC® 25922	Inhibited
<i>Staphylococcus aureus</i>	ATCC® 25923	Inhibited

Growth

TABLE OF SYMBOLS

LOT	Batch code	IVD	In vitro Diagnostic Medical Device		Manufacturer		Use by		Fragile, handle with care
REF	Catalogue number		Temperature limitation		Contains sufficient for <n> tests		Caution, consult instruction for use		Do not reuse

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LIOFILCHEM® S.r.l.