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E-Mail: liofilchem@liofilchem.net

Rev.2 / 03.09.2012

Biological indicators of H₂O₂ vapour superficial sterilization processes containing *Geobacillus stearothermophilus* (ATCC 7953) spores inoculated on special filter paper

DESCRIPTION

USP (United States Pharmacopoeia), EP (European Pharmacopoeia) and DAB (Deutsches Arzneibuch) standards recommend to use bioindicators during sterilization processes. Biological indicators **OXI CONTROL E6/E5 STRIP** are used for regular control of H₂O₂ vapour superficial sterilization cycles and validation of sterilizers. These biological indicators use a support (coupon) made by special filter paper. Strip has been inoculated with *Geobacillus stearothermophilus* (ATCC 7953) spores and has been introduced into a Tyvek®/Polytene envelope. Tyvek® side of the envelope is permeable to hydrogen peroxide vapours, while the Polytene side of the envelope is not permeable to these vapours. Biological indicators **OXI CONTROL E6/E5 STRIP** are produced under strictly controlled conditions in order to satisfy the requirements indicated in the USP and EP current editions and in accordance with ISO 11138 and EN 866 standards.

PRINCIPLE

OXI CONTROL E6/E5 STRIP contain *Geobacillus stearothermophilus* (ATCC 7953) spores in predefined concentrations: E6=1-5x10⁶ CFU/strip, E5=1-5x10⁵ CFU/strip. Spores are completely killed if the sterilization cycle has been efficient. In this case, during the following incubation in the Steri-Test Medium tubes, included in the package, spores are not able to grow and to modify medium's aspect. On the contrary, in the case the sterilization process has not been efficient, spores partially survive and therefore during the following incubation will be able to grow and to modify medium's aspect.

INSTRUCTIONS FOR USE

Before using biological indicators **OXI CONTROL E6/E5 STRIP**, allow them to reach room temperature (about 1 hour). Strips are exposed to vapours, during sterilization, inside their original envelope.

Exposure of indicators

Put envelopes containing coupons inside the sterilization environment in the points considered the most difficult to reach and to sterilize by hydrogen peroxide vapour.

Follow the operative instructions indicated by sterilizer manufacturer for the sterilization process.

NOTE: do not cover the Tyvek® side (white side) of the biological indicator.

Strips culture

At the end of sterilization/aeration process, remove the biological indicators and deliver them to the laboratory for the culture together with a not exposed control (positive control), belonging to the same batch

In optimal conditions all culture procedures have to be performed under a laminar flow cabinet following aseptic methods.

Submit to culture the biological indicators, opening the Tyvek®/Polytene envelope with a cooled sterile scissors and carefully transferring the strips inside a Steri-Test Medium tube, taking them by cooled sterile tweezers and letting them fall into the medium. Ensure that strips are completely dipped in the medium gently shaking the tube.

Incubation

Incubate the Steri-Test Medium tubes, that contain strips, at 55-60°C (131-140°F) for 7 days or for a shorter time validated by user.

EVALUATION

Geobacillus stearothermophilus (ATCC 7953) spores are killed off if the sterilization cycle has been efficient: in this case the medium's colour remains violet/clear even after incubation at 55-60°C (131-140°F) for the selected time.

If the sterilization cycle has not been efficient, spores partially survive and medium's colour turns yellow/turbid after incubation at 55-60°C (131-140°F) for the selected time.

The tube inoculated with the strip not submitted to the sterilization cycle and used as spore growth control (positive control), has to turn yellow/turbid after incubation. On the contrary, the test must be repeated after having investigated the causes of the negative result.

EVALUATION TABLE		
MEDIUM COLOUR	SPORE	STERILIZATION
Violet / clear	Killed off	Successful
Yellow / turbid	Vital	Unsuccessful

TREATMENT OF STERILTEST MEDIUM TEST TUBES AFTER USE

After use, sterilize the positive tubes (yellow/turbid) in autoclave at 121°C for at least 30 minutes and eliminate them in accordance with the procedures of the laboratory.

STORAGE

Store the product at 2-8°C: in these conditions it maintains its validity until the expiry date indicated on the label.

BIBLIOGRAPHY

1. United States Pharmacopoeia latest edition.
2. Deutsches Arzneibuch latest edition.
3. European Pharmacopoeia latest edition.
4. ISO 11138 and EN 866 latest edition.

PRESENTATION

PRODUCT	REF	PACKAGING	SPORES: CFU/strip	D _{VHP} (1.6 ± 0.5 mg/l, 30°C)
OXI CONTROL E6 STRIP	91088	20 strips + 20 Steri-Test Medium tubes	1-5x10 ⁶	1.5-4.0 minutes
OXI CONTROL E5 STRIP	91089	20 strips + 20 Steri-Test Medium tubes	1-5x10 ⁵	1.5-4.0 minutes

TABLE OF SYMBOLS

LOT	Batch code	 Do not reuse	 Manufacturer	 Use by	 Fragile, handle with care
REF	Catalogue number	 Temperature limitation	 Contains sufficient for <n> tests	 Caution, consult accompanying documents	

LIOFILCHEM® S.r.l.

Via Scozia Zona Ind.le - 64026 Roseto D.A. (TE) - Italy

Tel. +390858930745

Fax +390858930330

Website: www.liofilchem.net

E-Mail: liofilchem@liofilchem.net

Rev.2 / 03.09.2012